

# Instructions – Operating Shank's® table with 3-handle control valve



Your Shank's® Table is equipped with a 3-handle manual control valve that controls the hydraulic functions of the table. Simple instructions for operating the table are itemized below.

## US/North America

1. Connect grounded electrical cord (not included) to power source and connect cord to the 3-prong receptacle located on flip-lid covered electrical box mounted on the table base frame. **MAKE SURE CORD IS SECURELY CONNECTED.**
2. Flip toggle switch (located underneath electrical box) to turn on motor. **Motor must be running to perform any and all hydraulic functions on the table.**
3. Refer to Figure 1 for all functions of the table.
4. To disconnect cord, lift lid and grasp cord head to remove power cord from electrical box. **DO NOT GRASP AND REMOVE THE ELECTRICAL CORD FROM ANY DISTANCE AWAY FROM THE TABLE AS THIS MAY CAUSE DAMAGE TO THE ELECTRICAL BOX.**

## Export

1. Connect hard wired electrical power cord to power source. **MAKE SURE CORD IS SECURELY CONNECTED.**
2. Flip toggle switch (located on mounted electrical box) to the ON position to turn on motor. **Motor must be running to perform any and all hydraulic functions on the table.**
3. Refer to Figure 1 for all functions of the table

**Proper steps for locking casters and for retracting casters (to put table in stationary position)**

See separate CASTER OPERATION AND FUNCTION Instruction Sheet.

Figure 1

